

EZEEKLEEN 2.5

Distillation Alternative

Model: 9000

CHANGING YOUR CARTRIDGES

Tip: Only read monitor when the water is flowing through the Ezee Kleen 2.5

Step one: Obtain shrink wrapped 9001 DI Cartridge two pak. Remove cartridges from the shrink film.

Step two: One at a time remove the white dust plugs from either end of the cartridge, following the quick connect instructions given on page 17 of the users manual. The quick connect feature is on every fitting in the Ezee Kleen 2.5.

Step three: Turn off the supply water to the Ezee Kleen 2.5.

Step four: Disconnect the 90 degree elbows from each end of the 9001 DI Cartridges using the quick connect feature.

Step five: Snap the old 9001 DI Cartridges out of the brackets. Discard the old Cartridges.

Step Six: Snap the new 9001 DI Cartridges into the brackets.

→→→→ *Arrows on cartridge labels indicate the direction of water flow* →→→→

Step Seven: Reconnect the 90 degree elbows to each end of the 9001 DI Cartridges.

Step Eight: Turn supply water on to the Ezee Kleen 2.5 and run until there is a steady stream of water. The monitor should read below 5.00 and the red light will stop flashing.



16147 - 8A Avenue, Surrey, BC, V4A 8W8 • Ph: (800) 338-6693 • (604) 538-1285 • Fax: (604) 538 - 1295

© 2009-2019 Oasis Dental Group Inc.TM
www.oasisdentalgroup.ca


CONNECTING & DISCONNECTING CARTRIDGES

The Ezee Kleen 2.5 is equipped with a unique collet locking system that allows cartridge changes to take only seconds and requires no special training or tools.

To make a connection




1. Cut tube square

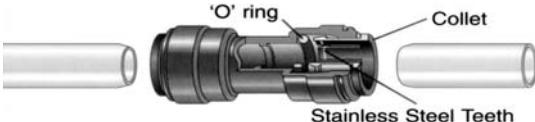


Cut tube square. It is essential that the outside diameter be free of score marks and that burrs and sharp edges be removed before inserting into fitting. For soft thin walled plastic tubing we recommend the use of a tube insert.

2. Insert tube

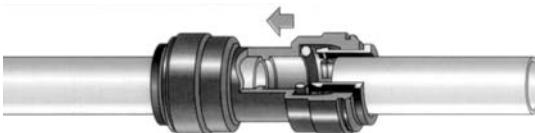

Fitting grips before it seals. Ensure tube is pushed into the tube stop.

3. Push up to tube stop

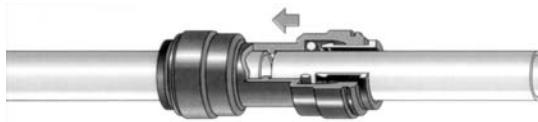

Push the tube into the fitting to the tube stop. The collet (gripper) has stainless steel teeth which hold the tube firmly in position while the 'O' ring provides a permanent leak proof seal.

4. Pull to check secure



Pull on the tube to check that it is secure. It is a good practice to test the system prior to leaving site and/or before use.

Disconnecting



Push in collet and remove tube




To disconnect, ensure the system is depressurized before removing the tube. Push in collet squarely against face of fitting, with the collet held in this position, the tube can be removed. The fitting can then be reused.

EZEEKLEEN 2.5


Distillation Alternative

Model: 9000

INTERNAL COMPONENTS DIAGRAM

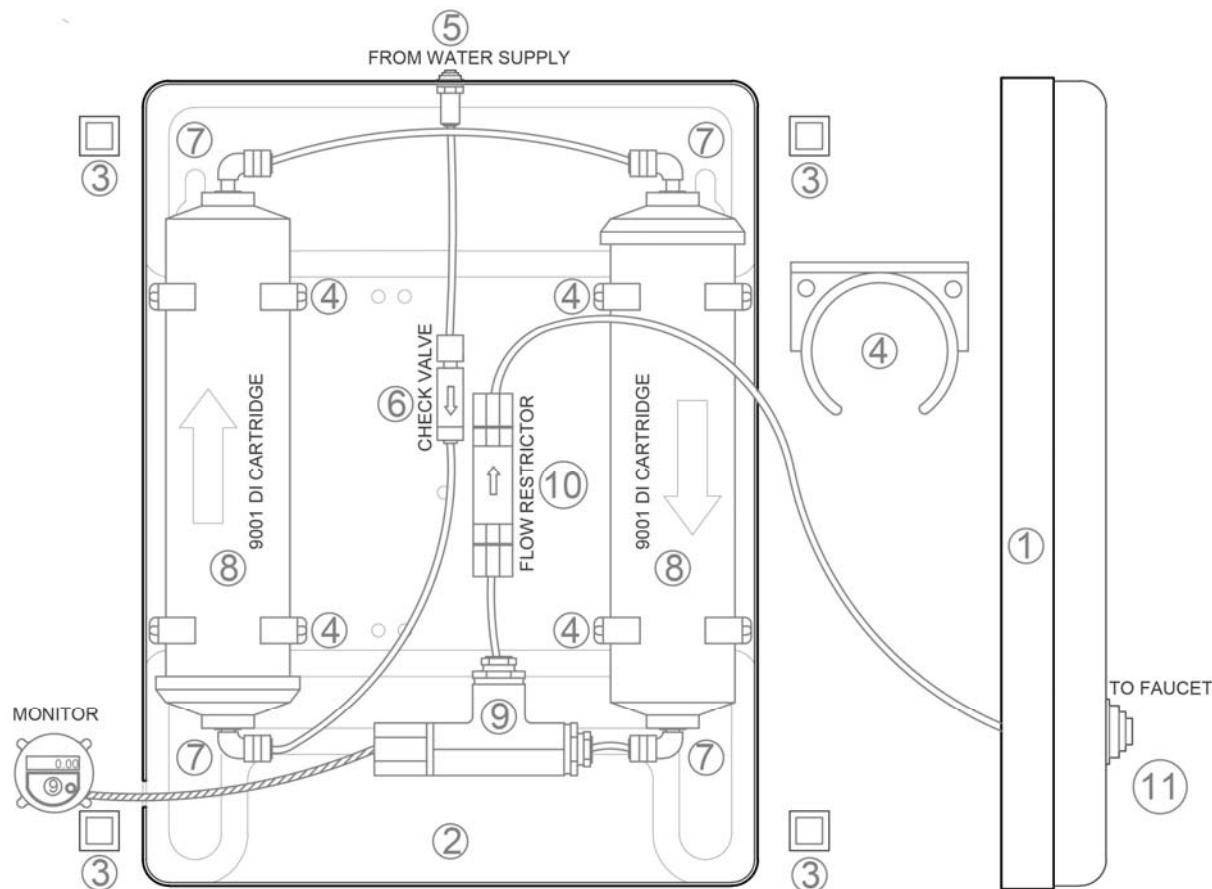

1. #9009 Ezee Kleen 2.5 Cover
2. #9009-1 Ezee Kleen 2.5 Case
3. #9011 Ezee Kleen 2.5 Bumpers
4. #9012 Ezee Kleen 2.5 Cartridge Clips
5. #9022 Ezee Kleen 2.5 1/4" X 1/4" Bulkhead Union
6. #9008 Ezee Kleen 2.5 Check Valve
7. #9006 Ezee Kleen 2.5 Slip in Elbow
8. #9001 Ezee Kleen 2.5 DI Cartridges
9. #9014 Ezee Kleen 2.5 Monitor
10. #9018 Ezee Kleen 2.5 Flow Restrictor
11. #9021 Ezee Kleen 2.5 1/4" X 3/8" Bulkhead Union
12. #9010 Ezee Kleen 2.5 Faucet Kit
13. #9027 Ezee Kleen 2.5 Diverter

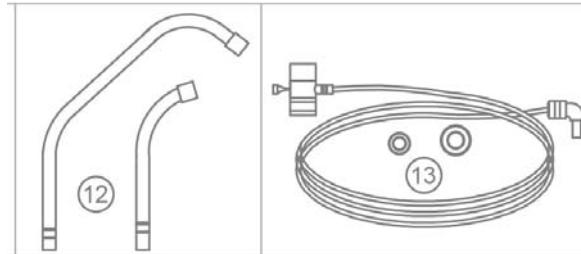

Oasis
DENTAL GROUP INC.

16147 - 8A Avenue, Surrey, BC, V4A 8W8 • Ph: (800) 338-6693 • (604) 538-1285 • Fax: (604) 538 - 1295