



## EXTENDED SHUTDOWN PROTOCOL

### **Ezee Kleen 2.5 Distillation Alternative (Model: 9000)**

#### **Under Counter Installation**

- Follow orange FEED line back to the cold water line, close the white shut-off valve with blue handle
- Open the dispensing faucet allowing residual water to drain from the system, close dispensing faucet once drained
- Unplug Monitor power supply

**If unit is out of use for longer than 2 weeks, REPLACE 9001 DI Cartridges**

**(& Optional 9001-XL XL DI Cartridge & Optional 9002 Sediment Cartridge)**

### **Ezee Kleen 2.5 HD 4-Stage Ultra Clean Water System (Model: 8000)**

- Follow orange FEED line back to the cold water line, close the white shut-off valve with blue handle
- Open the dispensing faucet allowing water to drain from the tank, close dispensing faucet once tank is drained (approximately 10-15 minutes) (water flow will stop)
- Follow yellow TANK line to the tank, close white shut-off valve with a blue handle
- Unplug Monitor & UV Lamp power supplies

**If unit is out of use for longer than 2 weeks, REPLACE 8003 Prefiltration Cartridge, 8004 RO**

**Membrane & 8002-FR DI Cartridge (& Optional 8002-XL XL DI Cartridge)**

Questions? Contact customer support Mon – Fri, PST, 8:00am – 5:00pm.



16147 - 8A Avenue, Surrey, BC, V4A 8W8 • Ph: (800) 338-6693 • (604) 538-1285 • Fax: (604) 538 - 1295